# PULPDENT® Embrace™ First-Coat

## PRODUCT DESCRIPTION

Embrace First Coat is a unique one-step resin-based primer that bonds chemically and mechanically to etched ceramic and abraded metal surfaces. It is an optional enhancer in combination with Dental Cements for adhesion and sealing, particularly in challenging clinical conditions involving moisture.

## INDICATIONS

Cement enhancer for luting direct and indirect restorations.

## CONTRAINDICATIONS

Do not use on patients with known resin allergies. For any potential allergies, refer to associated MSDS.

## INSTRUCTIONS FOR USE

1. Pre-treat the restoration surfaces accordingly.
2. Apply a thin coat of Embrace First-Coat on the clean dry metal and/or ceramic surfaces with a flocked applicator tip.
3. Light cure for 20 seconds.

## WARNING:

- Embrace First-Coat may cause eye or skin irritation on contact.
- Always wear protective gloves and eyewear when using Embrace First-Coat.
- In case of eye contact with Embrace First-Coat, flush thoroughly with water at a sink or emergency eye wash station.

## STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.

## REPORTING OF SERIOUS INCIDENTS


Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262 / [www.pulpdent.com](http://www.pulpdent.com) / email: [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

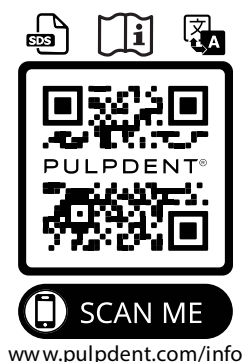SDS	Electronic Safety Data Sheet
EN	Electronic Safety Data Sheet
ES	Ficha electrónica de datos de seguridad
FR	Fiche de données de sécurité électronique
DE	Elektronisches Sicherheitsdatenblatt
IT	Scheda di sicurezza elettronica
DK	Elektronisk sikkerhedsdatablad
NL	Elektronisch veiligheidsinformatieblad
SE	Elektroniskt säkerhetsdatablad
NO	Datablad om elektronisk sikkerhet
FI	Sähköinen käyttöturvallisuustiedote
CZ	Elektronický bezpečnostní list
LV	Elektroniskā drošības datu lapa
LT	Elektroninis saugos duomenų lapas
PT	Ficha de dados de segurança eletrónica
SK	Elektronická karta bezpečnostných údajov
PL	Elektroniczna karta charakterystyki
RO	Fișa electronică cu date de securitate
EE	Elektroniline ohutuskaart
BG	Електронен информационен лист за безопасност
GR	Ηλεκτρονικό δελτίο δεδομένων ασφαλείας

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**NOTIFICATION DES INCIDENTS GRAVES**

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