PRODUCT DESCRIPTION

Embrace WetBond Seal-n-Shine is an unfilled resin specifically designed for sealing, finishing and polishing composite restorations. It penetrates and seals porosities and cracks in composites, seals margins, and provides quick finishing and polishing of composite, etched or abraded ceramic restorations, and acrylic temporary crowns.

INDICATIONS

Indicated for sealing, finishing and polishing composite restorations, and for repairing cracked teeth, sealing of non-caries or pre-caries lesions, enamel defects and notches, and for sealing the margins of indirect restorations cemented with resin cement.

CONTRAINDICATIONS

Not indicated for direct placement on the exposed pulp.

INSTRUCTIONS FOR USE

Dry ceramic, metal, resin desensitized, prehybridized, and cured resin surfaces. Leave prepared tooth surface slightly moist. Slightly moist surfaces exhibit neither dryness nor pooling of water. Lightly dry and remove surface water with compressed air or a cotton pellet, but do not desiccate the tooth, which naturally contains water. Tooth surface should be shiny or glossy. Overly wet surfaces will result in decreased bond strength.

- For enamel and composite, etch or microabrade for 10 seconds. For ceramic surfaces, place rubber dam and etch with Pulpdent Porcelain Etch Gel for 2 minutes. Pulpdent Kool-Dam should be used to protect soft tissue, teeth or porcelain surfaces, as required.
- Rinse well. Dry ceramic, metal, resin desensitized, prehybridized and cured resin surfaces. Lightly dry enamel and leave prepared tooth surface slightly moist.
- When necessary, place dental floss through the contacts and leave it in the embrasures. Use later to clear contacts and prevent bonding to adjacent tooth surfaces. The teeth can be wedged and a mylar strip can be placed, if desired.
- Apply a thin coat of Seal-n-Shine to the etched tooth and restorative surfaces with a flocked dispenser tip or a clean brush.
- Remove the dental floss by lifting interproximally through the contacts. It is important for the contacts to be open.
- Light cure. Seal-n-Shine cures with all lights.

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multi-ple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to: Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com
Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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**PULPDENT® Corporation**
80 Oakland Street • Watertown, MA 02472 • U.S.A.