


FILE-RITE™

Canal Lubricant and Chelating Agent

 4 x 5 gm File-Rite



Rx only

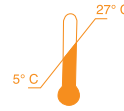

PULPDENT

FILE-RITE™
CANAL LUBRICANT AND CHELATING AGENT

CE 0459

File-Rite is a 17% EDTA gel that has been buffered to a neutral pH. During instrumentation and debridement of the root canal, File-Rite is an effective calcium binding material for chelating and removing dentin from the canal walls, and it acts as a lubricant to minimize the binding and breaking of files.

Directions for Use

1. Always place a rubber dam.
2. Remove cap and securely attach a 30 gauge x 1" applicator tip to syringe.
3. Dispense File-Rite directly into the canal. For controlled dispensing, hold syringe barrel with fingertips, place plunger in palm, and gently pull back with fingers. File-Rite can also be placed directly on the file or into a dappen dish.
4. Instrument the canal. Reapply File-Rite and repeat as necessary until the canal is enlarged and shaped as desired.
5. Rinse canal thoroughly.
6. Discard needle and place cap back on the syringe. Use a new needle for each patient.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to:

<http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>.

PULPDENT Corporation

80 Oakland St. Watertown, MA 02472 USA

Tel: (800) 343-4342 / (617) 926-6666 Fax: (617) 926-6262

www.pulpdent.com email: pulpdent@pulpdent.com

XP-FILE-IN-06w
REV: 04/2018