PULPDENT®

Spee-Dee™ Build-Up

Multi-Purpose Core and Tooth Build-Up Resin

Dual Cure • Fluoride Releasing • Hydrophilic • Radiopaque

Spee-Dee Build-Up is a multi-purpose, dual cure, resin-based dental restorative and core build-up material. It is moisture friendly and has self-adhesive and self-etching properties that are compatible with all bonding agents. The material is fluoride releasing and radiopaque. It contains no Bisphenol A, no Bis-GMA, and no BPA derivatives.

Spee-Dee Build-Up is dense like dentin and therefore cuts like dentin. Rotary instruments transition smoothly from tooth to Spee-Dee, cut accurately, and do not ditch, gouge, gum up or chatter.

Spee-Dee Build-Up is used for core build-up, post cementation, and as a multi-purpose repair material for fractured cusps and broken down teeth, defects in crown preparations, lost partial or whole fillings, and chipped or broken crowns or bridges.

Posts and cores can be placed in one step ensuring a homogeneous, one-piece internal structure.

In restorations with retention form, Spee-Dee Build-Up's self-adhesive and self-etching properties may be sufficient for post and core procedures without the use of a bonding agent. However, to achieve highest bond strengths, etching and bonding agents are recommended.

PRODUCT DESCRIPTION

Spee-Dee Build-Up is a multi-purpose, dual cure, resin-based dental restorative and core build-up material. It is moisture friendly and has self-adhesive and self-etching properties that are compatible with all bonding agents. The material is fluoride releasing and radiopaque. It contains no Bisphenol A, no Bis-GMA, and no BPA derivatives.

INDICATIONS

Multi-purpose, resin-based dental restorative and core build-up material

CONTRAINDICATIONS

Not indicated for direct placement on the exposed pulp.

HOW TO USE THE DOUBLE BARREL CARTRIDGE:

1. Remove cap. If necessary bleed the cartridge so that base and catalyst are at the orifice of the barrels. Place a mixing tip on the double barrel cartridge.
2. To ensure an even mix of base and catalyst, dispense 2-3 mm of material onto a pad and discard this material.
3. Dispense evenly mixed material directly onto the tooth or into the restoration.
4. Discard mixing tip. Recap cartridge. Do not cross-contaminate base and catalyst.

INSTRUCTIONS FOR USE

1. Remove the old restoration and/or decay, or prepare post space using post system of choice
2. To achieve highest bond strengths, etch and place bonding agent according to the manufacturer's instructions. Light cure. A dual cure bonding agent is recommended with self-cure and dual cure resin core materials..
3. Remove cap from double barrel cartridge. If necessary bleed the cartridge so that base and catalyst are at the orifice of the barrels.
4. Place a mixing tip on the double barrel cartridge and insert into an appropriate 1:1 cartridge dispenser. To ensure an even mix of base and catalyst, dispense 2-3 mm of material onto a pad and discard this material.
5. Dispense material into the preparation site, or dispense into the post hole and place the post. Light cure for 20 seconds. Build-up the core using "Crown Up" technique by placing material, curing, adding more material, curing, and repeating this until the desired build-up is accomplished. The material will also self-cure. Anaerobic self-cure setting time at 37°C is 3:15 minutes. Carefully recap cartridge. Do not cross-contaminate base and catalyst.
6. Prepare build-up to receive the restoration.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, cross-contamination between parts, and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.